

#8400 ACUTY PROJECTOR SLIDE INSTRUCTIONS

The Stereo Optical Acuity Test Slide has been designed for use with Marco, A.O. (Reichert), Woodlyn and similar projectors.


CONVERT FOR USE IN A 10 FOOT LANE

Although initially configured for use in a 20 foot lane, the 8400 test slide can be easily converted for a 10 foot lane.

1. The glass slide is held in the frame by a spring in the bottom channel.
2. Remove the glass slide from the frame as follows:
 - ◆ Grasp slide with one hand and frame with the other.
 - ◆ Slide glass down within frame.
 - ◆ Tilt top of slide out and remove.
3. Rotate glass slide on a vertical axis.
4. Return slide to frame by reversing process step 2.

CLEANING INSTRUCTIONS

Do not immerse slide in any liquid or spray liquid directly on slide, use a soft damp cloth to clean.

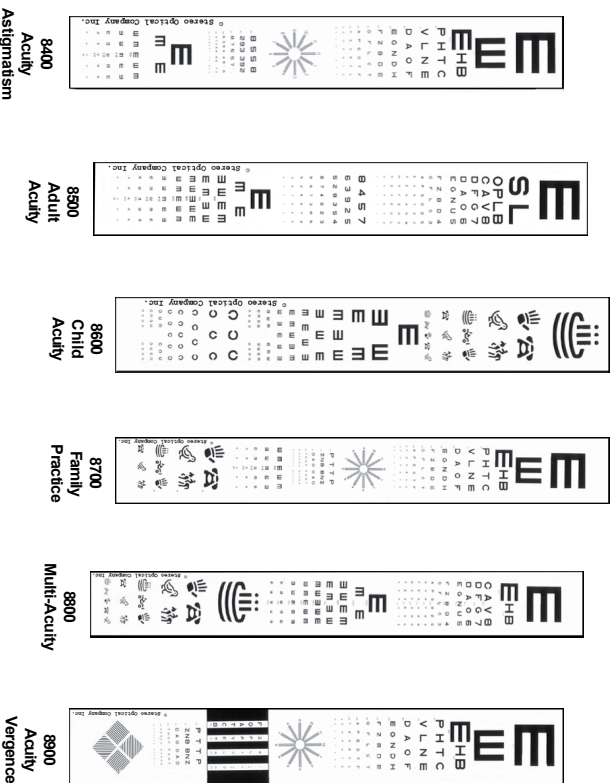

#8400 ACUTY PROJECTOR SLIDE ANSWER KEY

LETTER ACUTY

| LEVEL | ANSWER |
|--------|---------------|
| 20/400 | RIGHT |
| 20/300 | UP |
| 20/200 | E |
| 20/100 | H B |
| 20/80 | P H T C |
| 20/70 | V L N E |
| 20/60 | D A O F |
| 20/50 | E G N D H |
| 20/40 | F Z B D E |
| 20/30 | O F L C T |
| 20/25 | A P E O T F |
| 20/20 | T Z V E C L |
| 20/15 | O H P N T Z |
| 20/10 | E V O T S H F |

MIRROR KEY NUMBER ACUTY

| LEVEL | ANSWER |
|-------|---------------------------|
| 20/50 | 8 5 - 5 8 |
| 20/40 | 2 9 3 - 3 9 2 |
| 20/30 | 8 7 5 - 5 7 8 |
| 20/20 | 6 3 9 4 - 4 9 3 6 |
| 20/15 | 3 7 2 6 8 - 8 6 2 7 3 |
| 20/10 | 3 2 8 5 3 7 - 7 3 5 8 2 3 |

TUMBLING "E" ACUTY

| LEVEL | ANSWER |
|--------|---------------------------------|
| 20/200 | RIGHT |
| 20/100 | BOTTOM |
| 20/50 | TOP BOTTOM RIGHT TOP BOTTOM |
| 20/40 | LEFT TOP BOTTOM RIGHT LEFT |
| 20/30 | RIGHT LEFT TOP BOTTOM RIGHT |
| 20/20 | LEFT BOTTOM RIGHT BOTTOM BOTTOM |
| 20/15 | RIGHT TOP LEFT BOTTOM LEFT |



STEREO OPTICAL CO., INC.
8623 W. BrynMawr Ave., Suite 502
Chicago, IL 60631
800-344-9500
Fax 773-867-0388
www.stereooptical.com
sales@stereooptical.com